# LOWER LEG SUPPORT

AM3000

## INSTRUCTIONS FOR USE

Issue Date: 05/2026

Revision: F

**EC Representative**  
Qualrep Services B.V.  
Utrechtseweg 310 Bldg B42  
6812 AR Arnhem  
Netherlands

**Manufacturer**  
Adept Medical Ltd  
2-6 McDonald St,  
Morningside  
Auckland 1025, New Zealand

[adeptmedical.com](https://adeptmedical.com) |

ADEPT

Medical  
Supporting you

**Product Registration**

Register today to ensure your product is covered by warranty, and for easy access to product services and general support.

[www.adeptmedical.com/contact/product-registration](http://www.adeptmedical.com/contact/product-registration)

**Warranty**

**Service**


**Support**

**Contents**


Unboxing	<b>4</b>		
Assembly	<b>4</b>		
Setup	<b>4</b>		
Spare Parts	<b>7</b>		
Image Table Verification	<b>7</b>		
Essential Information	<b>8</b>		
<hr/>			
ES   Spanish	<b>10</b>	NO   Norwegian	<b>32</b>
FR   French	<b>12</b>	PL   Polish	<b>34</b>
IT   Italian	<b>14</b>	PT   Portuguese	<b>36</b>
DE   German	<b>16</b>	RO   Romanian	<b>38</b>
NL   Dutch	<b>18</b>	RU   Russian	<b>40</b>
HR   Croatian	<b>20</b>	SR   Serbian	<b>42</b>
CS   Czech	<b>22</b>	SK   Slovak	<b>44</b>
DA   Danish	<b>24</b>	SL   Slovenian	<b>46</b>
FI   Finnish	<b>26</b>	SV   Swedish	<b>48</b>
EL   Greek	<b>28</b>	TR   Turkish	<b>50</b>
HU   Hungarian	<b>30</b>		

## Unboxing


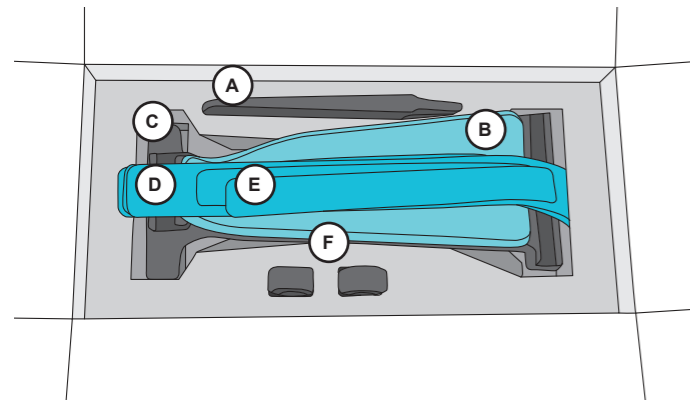


	(A)	Foot Plate x 1
	(B)	Leg Pad x 1
	(C)	Lower Leg Support x 1
	(D)	Leg Strap x 2
	(E)	Foot Strap x 1
	(F)	Table Strap x 2

## Assembly
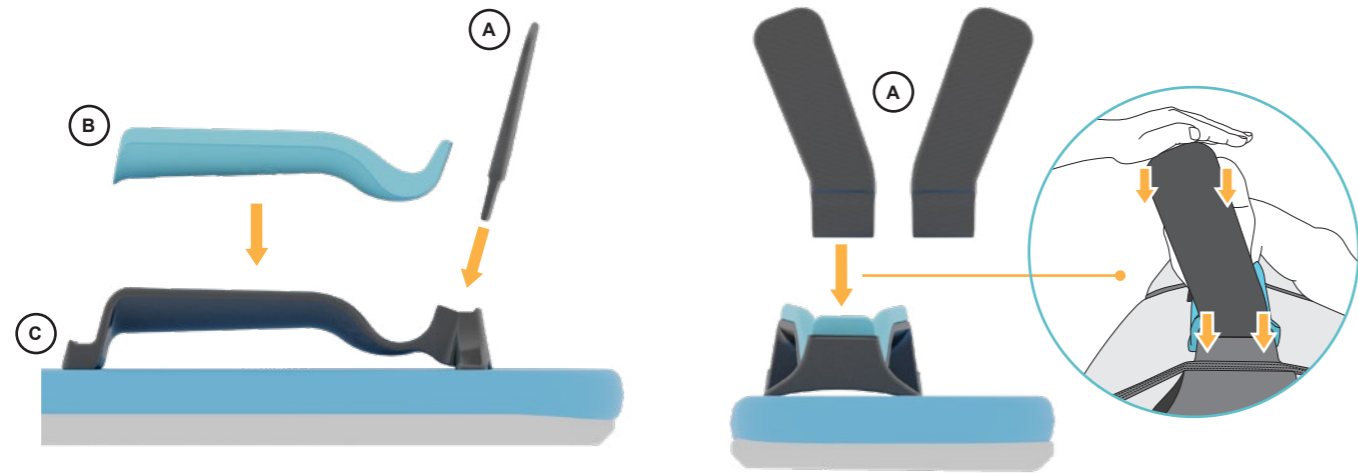
## Setup

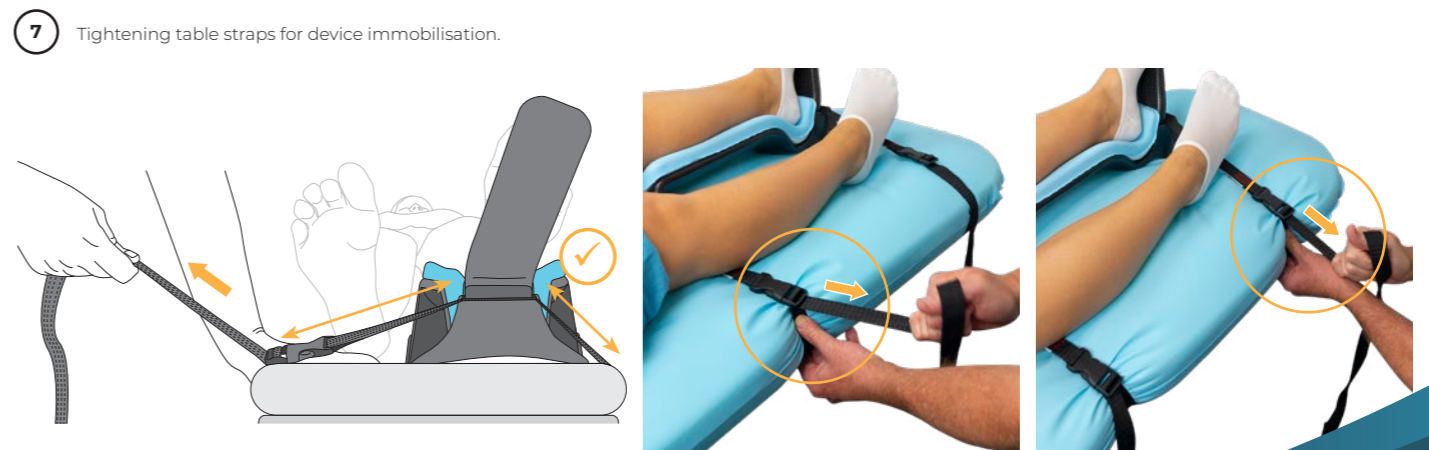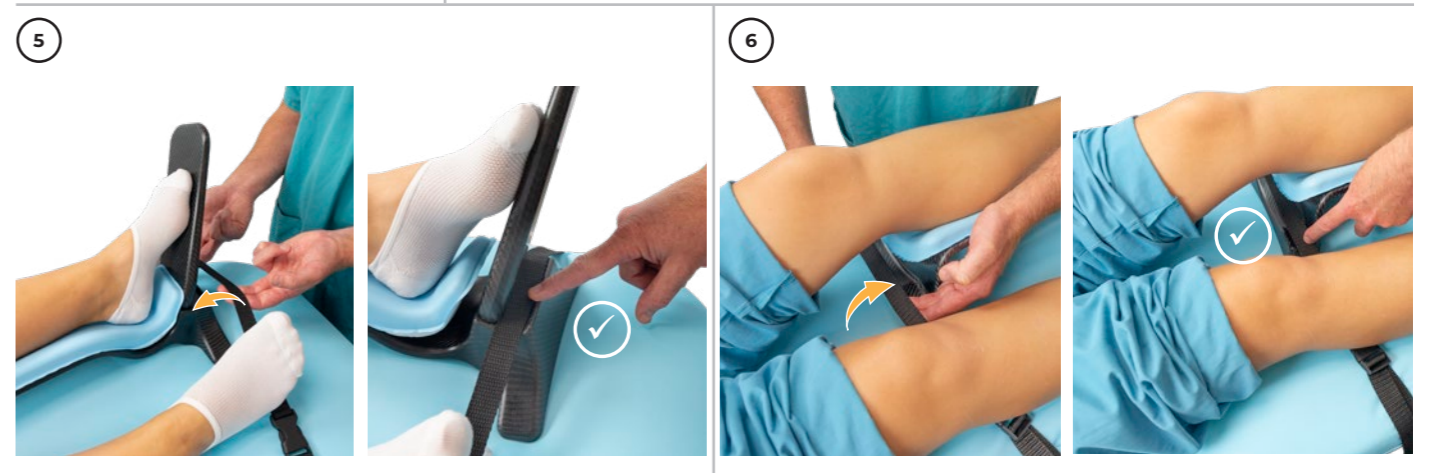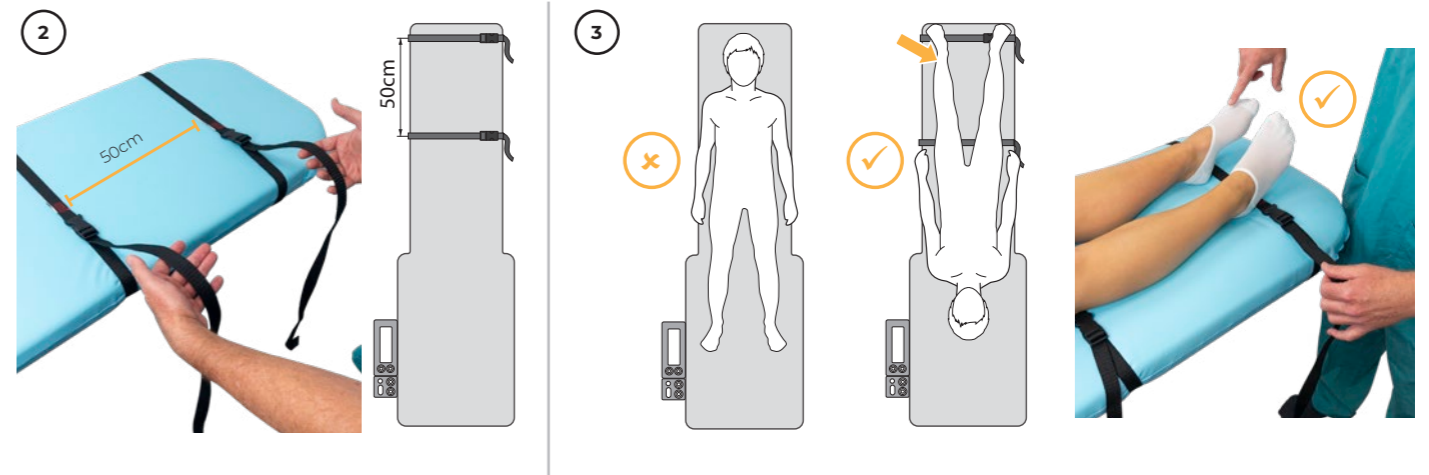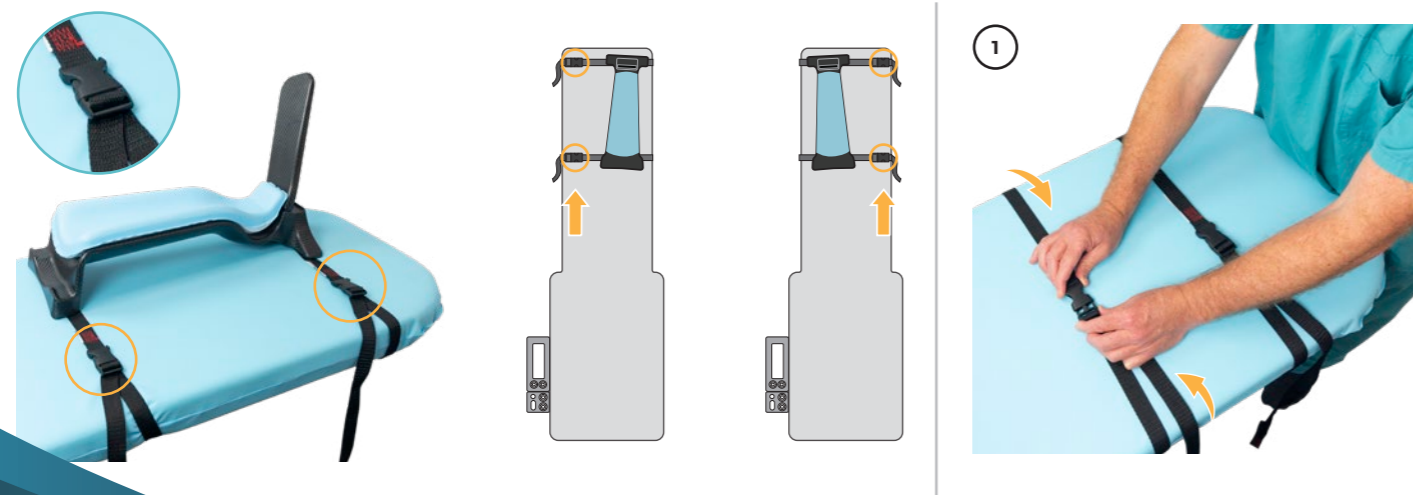**Foot Plate Removal**

**Spare Parts**


**Product Code**

	Lower Leg Support	AM3000
---	-------------------	--------

**Component Codes**

	(A)	Foot Plate	M6511
	(B)	Leg Pad	M6512
	(C)	Support Base	M6516
	(D)	Leg Strap	M6513
	(E)	Foot Strap	M6514
	(F)	Table Strap	M6515

**Image Table Verification**

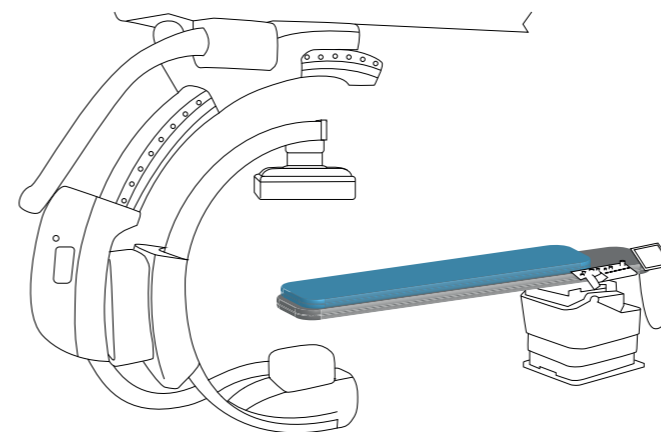The product has been risk assessed to operate within the below specifications

**Table Angulation:** +/- 15° lateral & longitudinal  
**Table Surface:** Flat

## Essential Information

### Information

#### Intended Purpose

To support and immobilise a supine patient's leg during endovascular surgery.

#### Intended User/Training Requirement

Intended to be used by trained medical professionals.

#### Patient Target Group

Adult male or female patient requiring diagnostic imaging and image guided therapy.

#### Contraindications

- Not to be used with patients over 135 kg.
- Not for paediatric use.

#### Warnings and Cautions

- Ensure IFU is read prior to use.
- Ensure Preparatory Cleaning is conducted.
- Ensure Preparatory Product Check is conducted.
- Do not over tighten Leg & Foot Straps.
- Do not use with patients over 135 kg.

#### Incident Reporting Guidelines

For product complaints and incidents, please complete form on: [www.adeptmedical.com/contact/service-and-repairs](http://www.adeptmedical.com/contact/service-and-repairs)

### Initial Checks & Cleaning

#### Cleaning

Refer to Disinfection Instructions.

#### Product Checks

- No break in Carbon Fibre surface.
- No visible damage or sharp edges, e.g. cracks.
- Straps: ensure there is no damage, e.g. tearing.
- Table Strap: ensure buckle can fully engage and disengage smoothly.
- Table Strap: ensure Table Strap is secure and correctly placed (page 5).
- Foot/Leg Strap: ensure straps are secure and correctly placed (page 6).
- Foot Plate: Ensure the Foot Plate is correctly placed in the Foot Plate Slot and fully engaged.

### Disinfection

#### Warnings

- Insufficient cleaning may compromise disinfection process and lead to transmission of pathogens.
- Do not use any disinfectant products or abrasive/corrosive agents which are not on the approved cleaners list.
- Always read manufacturer's instructions and consult the manufacturers MSDS for cleaning and disinfectant products.
- Do not use any cleaning procedure which exceeds 65°C/149°F.

#### Limitations on Processing

Discontinue use if:

- Any cracks or breakages are present.
- Any exposed carbon fibre is visible.
- The Foot Plate does not securely fit into the Lower Leg Support Base.
- Any tearing is observed on the Leg Pad or any of the Straps.
- The hook & loop on the Leg/Foot Strap does not securely engage.
- The buckle on the Table Strap does not securely engage.

### Disinfection Instructions

#### Initial treatment at the point of use

It is important to clean the product once it is removed from the packaging and after each procedure. Ensure all areas including joints, clips, sockets, brackets and levers are thoroughly cleaned after use to remove all contaminant build-up that may be present following a procedure.

#### Preparation before cleaning

- Lower Leg Support Base: Remove the base from the mattress.
- Foot Plate: Detach the Foot Plate from the Base.
- Leg Pad: Remove the padding from the Base.
- Leg Strap: Unfasten the strap i.e. ensure the "hook" and "loop" sides are not connected.
- Foot Strap: Unfasten the strap i.e. ensure the "hook" and "loop" sides are not connected.
- Table Strap: Unfasten the buckle on the strap.

[www.adeptmedical.com/imaging-table-accessories/lower-leg-support/training?group=cleaning-instructions](http://www.adeptmedical.com/imaging-table-accessories/lower-leg-support/training?group=cleaning-instructions)

#### Cleaning: Manual

Lower Leg Support Base/Foot Plate

- If visibly soiled rinse with water.
- These components can be fully submerged in water.
- A soft bristled brush can be used if required.
- Ensure soiling is fully removed.
- Ensure all components are completely dry prior to disinfection.

Leg Pad/Leg Strap/Foot Strap/Table Strap

- If visibly soiled rinse with water.
- These components should not be fully immersed in water.
- A soft bristle brush can be used if required.
- Ensure soiling is fully removed.
- Ensure all components are fully dry prior to disinfection.

#### Disinfection

Refer to the Approved Cleaners List for selection of an appropriate disinfecting agent.

Lower Leg Support:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including inside the Foot Plate slot.

Foot Plate/Leg Pad/Leg Strap/Foot Strap and Table Strap:

- With a cloth dampened by an approved disinfectant, wipe all surfaces.

### Approved Cleaners List

#### Approved Disinfectant by Brands

##### Liquids:

ORION Laboratories - 70% Isopropyl Alcohol  
Jaychem Industries - 2% Chlorhexidine / 70% Alcohol  
Betadine - Povidone Iodine (7.5%)  
BODE Chemie GmbH - Kohrsolin FF  
STERIS - Coverage Spray TB  
CaviCide - Metrex Research

##### Dissolvable:

Du Pont - Rely+On Virkon Tablets

##### Surface Wipes:

PDI Sani-Cloth Bleach  
PDI Sani-Cloth AF3  
Reynard Health Supplies - Surface Disinfectant Wipes  
Metrex Research - CaviWipes  
Clinell Wipes - Universal (Green)  
Clorox Hydrogen Peroxide Wipes

#### Approved Active Disinfectant Ingredients

##### Acid Based:

≤10% Malic Acid CAS 6915-15-7  
≤6% Sulfamic acid CAS 5329-14-6

##### Alcohol Based:

≤5% 2-Butoxyethanol CAS 111-76-2  
≤10% Butyldiglycol CAS 112-34-5  
≤70% Isopropyl Alcohol (Propan-2-ol) CAS 67-63-0  
≤10% Tridecanol CAS 69011-36-5  
<10% Alcohols, C12-14, ethoxylated CAS 68439-50-9  
≤70% Denatured Ethanol CAS 64-17-5

##### Sulphate Based:

≤0.1% PHMB CAS 27083-27-8  
≤55% Potassium Peroxymonosulfate CAS 70693-62-8  
≤3% Potassium Persulfate CAS 7727-21-1

##### Ammonium and Chloride Based:

≤5% Benzalkonium Chloride CAS 68424-85-1  
≤0.28% Benzethonium Chloride CAS 121-54-0  
≤10% Benzyl-C23-18-Alkyl-dimethyl Ammonium Chloride CAS 8001-54-5  
≤2% Chlorhexidine CAS 55-56-1  
≤10% Didecyl Dimethyl Ammonium chloride CAS 7173-51-5  
≤0.5% Quaternary Ammonium Compounds CAS 68956-79-6  
<5% Benzyl-C 12-18 alkyldimethylammonium chlorides CAS 63891-01-5

##### Others:

≤10% Glutaral CAS 111-30-8  
≤0.63% Sodium Hypochlorite CAS 7681-52-9  
≤7.5% Povidone Iodine CAS 25655-41-8  
0.5% Disodium Cocoampho Dipropionate CAS68604-71-7  
≤1.4% Hydrogen Peroxide CAS 7722-84-1

Access the latest Approved Cleaners list via [www.adeptmedical.com/resources](http://www.adeptmedical.com/resources)

### Drying

All components should be dried thoroughly before use.

### Maintenance, Inspection and Testing

Lower Leg Support/Foot Plate:

- No exposed Carbon Fibre is visible.
- No visible damage or sharp edges, e.g. cracks.
- Ensure that the Foot Plate securely fits in to Lower Leg Support.

Leg Pad:

- No visible damage or tearing.

Foot Strap/Leg Strap:

- No visible damage or tearing.
- The hook & loop should attach securely.

Table Strap:

- No visible damage or tearing.
- The buckle can fully engage and disengage smoothly.

### Packaging

It is not required to package the device following disinfection.

### Sterilisation

This device should not be subjected to sterilisation processes.

### Storage

Once disinfection is complete and all components are dry the device should be stored in a dry environment.

### Disposal

Used product is a biohazard, decontaminate according to instruction provided in the Disinfection Instructions and reach out to the manufacturer on [adeptmedical@adept.co.nz](mailto:adeptmedical@adept.co.nz) for material specification, if required. Dispose according to internal clinic policy taking into consideration local regulations.

### Serious Incidents

Any serious incident which occurs in relation to the device should be reported immediately to:

- The Competent Authority of the applicable Member State.
- The Manufacturer using the Serious Incident Reporting Form found on the Adept Medical Website: [www.adeptmedical.com/contact/serious-incident-reporting-form](http://www.adeptmedical.com/contact/serious-incident-reporting-form)

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Visit the Adept Medical Website for detailed visual instructions related to this device. [www.adeptmedical.com](http://www.adeptmedical.com)

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