RETROGRADE IR PLATFORM

AM0420 · AM0400 · AM0410
AM0460 · AM0450 · AM0470

INSTRUCTIONS FOR USE

Issue Date: 03/2026

Revision: G

EC Representative
Qualrep Services B.V.
Utrechtseweg 310 Bldg B42
6812 AR Arnhem
Netherlands

Manufacturer
Adept Medical Ltd
2-6 McDonald St,
Morningside
Auckland 1025, New Zealand

adeptmedical.com |

ADEPT

Medical
Supporting you

Product Registration

Register today to ensure your product is covered by warranty, and for easy access to product services and general support.

www.adeptmedical.com/contact/product-registration

Warranty

Service


Support

Contents

Unboxing	4	
Assembly	4	
Setup	5	
Spare Parts	7	
Image Table Verification	7	
Essential Information	8	
<hr/>		
ES Spanish	10	NO Norwegian 32
FR French	12	PL Polish 34
IT Italian	14	PT Portuguese 36
DE German	16	RO Romanian 38
NL Dutch	18	RU Russian 40
HR Croatian	20	SR Serbian 42
CS Czech	22	SK Slovak 44
DA Danish	24	SL Slovenian 46
FI Finnish	26	SV Swedish 48
EL Greek	28	TR Turkish 50
HU Hungarian	30	

Unboxing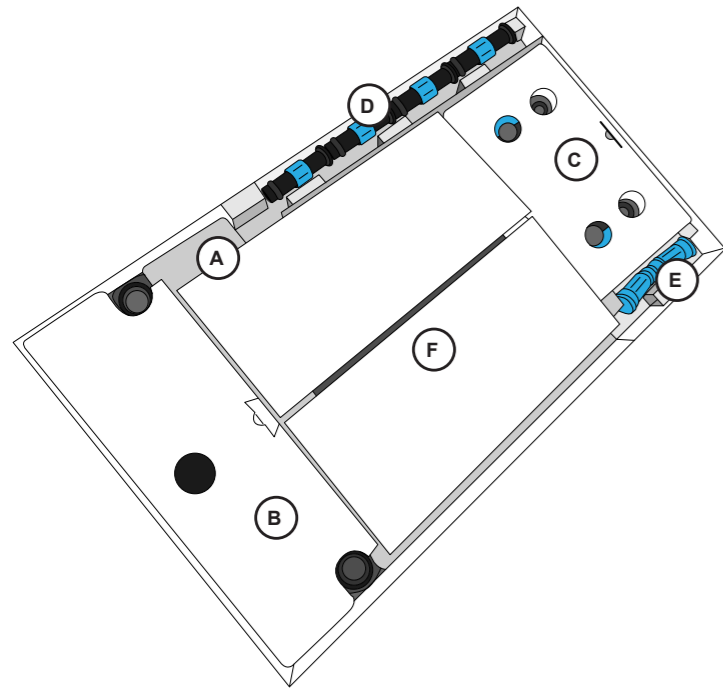

	(A)	Retrograde Main Platform (Narrow) x1
	(B)	Front Base (Narrow) x1
	(C)	Rear Base x1
	(D)	Leg x 4
	(E)	Leg Extension x 2
	(F)	Extension (Narrow) x1

AM0400 / AM0450

(A) (B) (C) (D) (E)

AM0410 / AM0470


(F)

AM0420 / AM0460


(A) (B) (C) (D) (E) (F)

Assembly

(1)

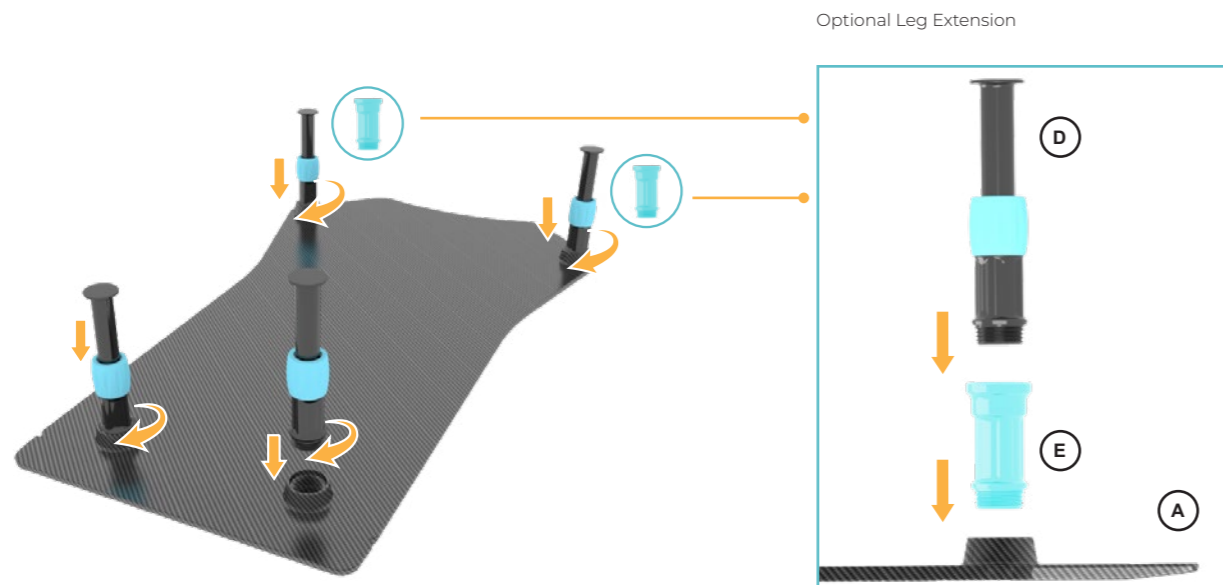
(2)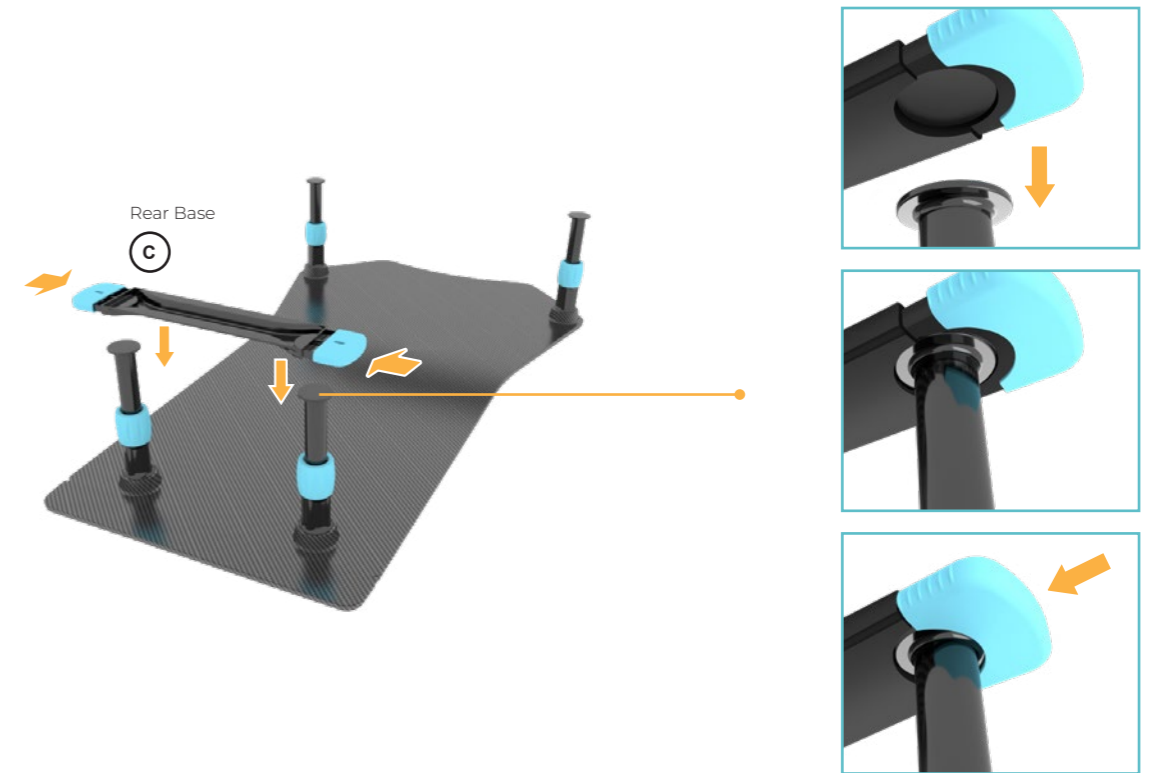


Setup


(1)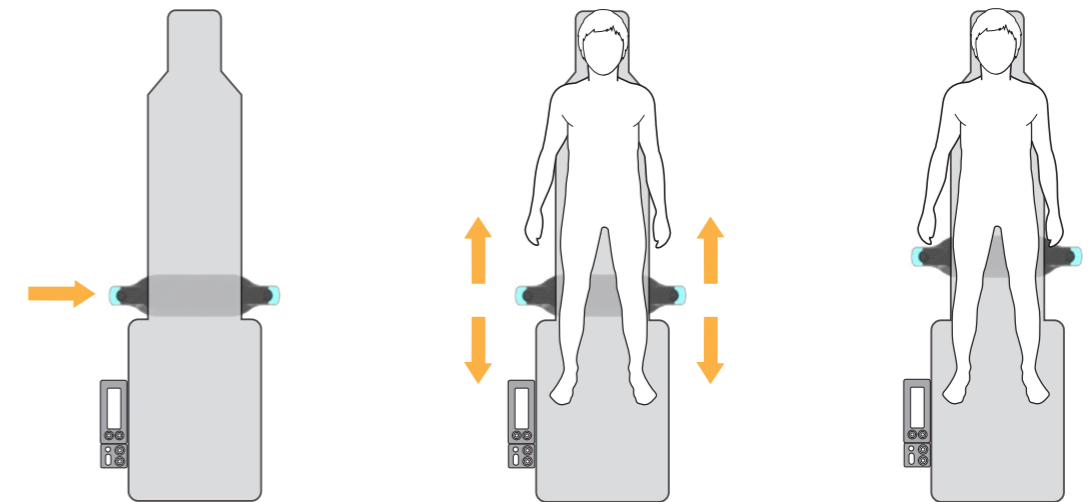


(2)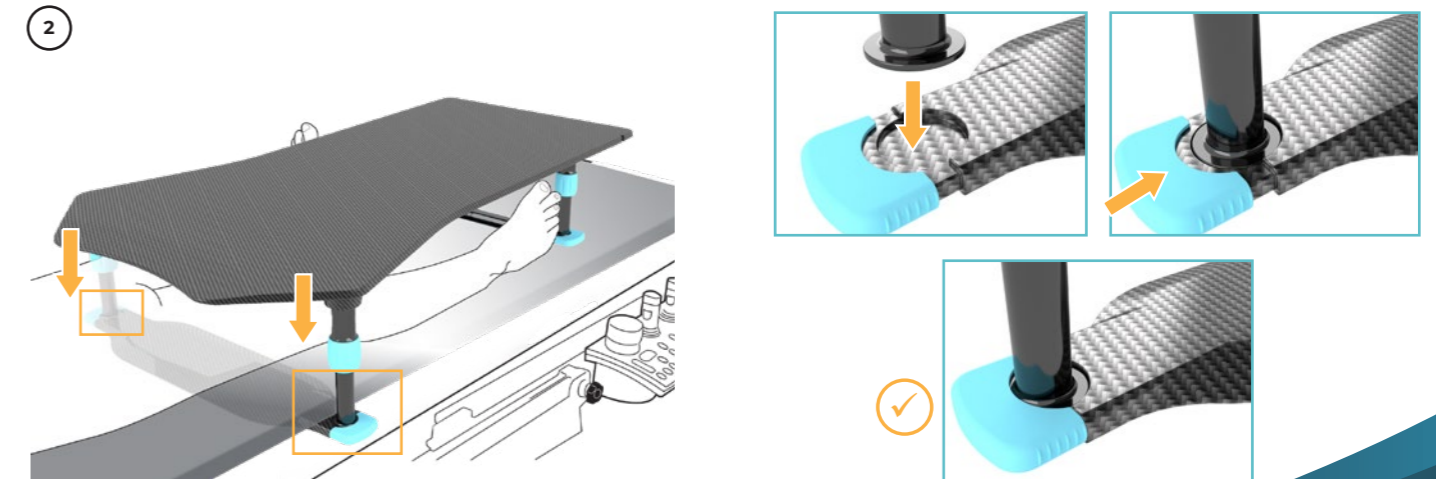


3

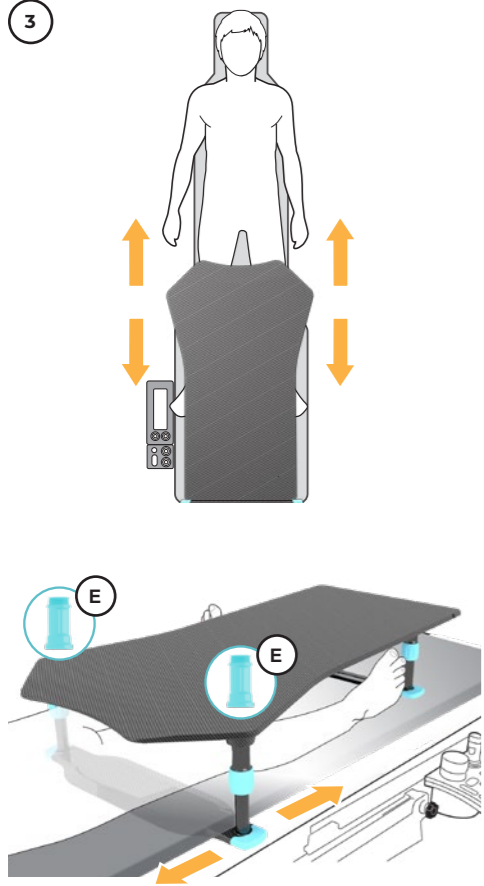

4 Unlock the leg to adjust the platform height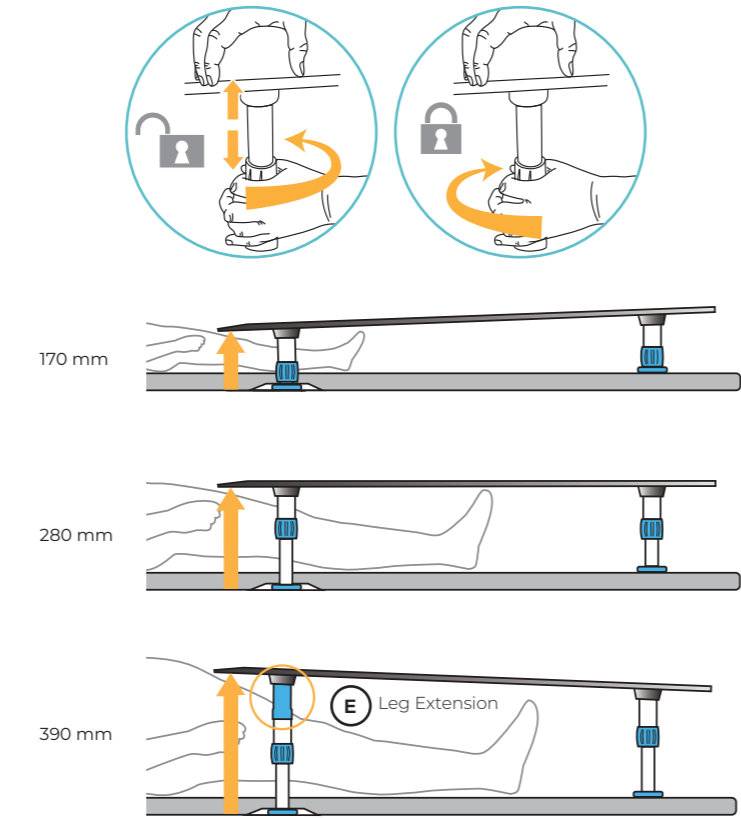

5

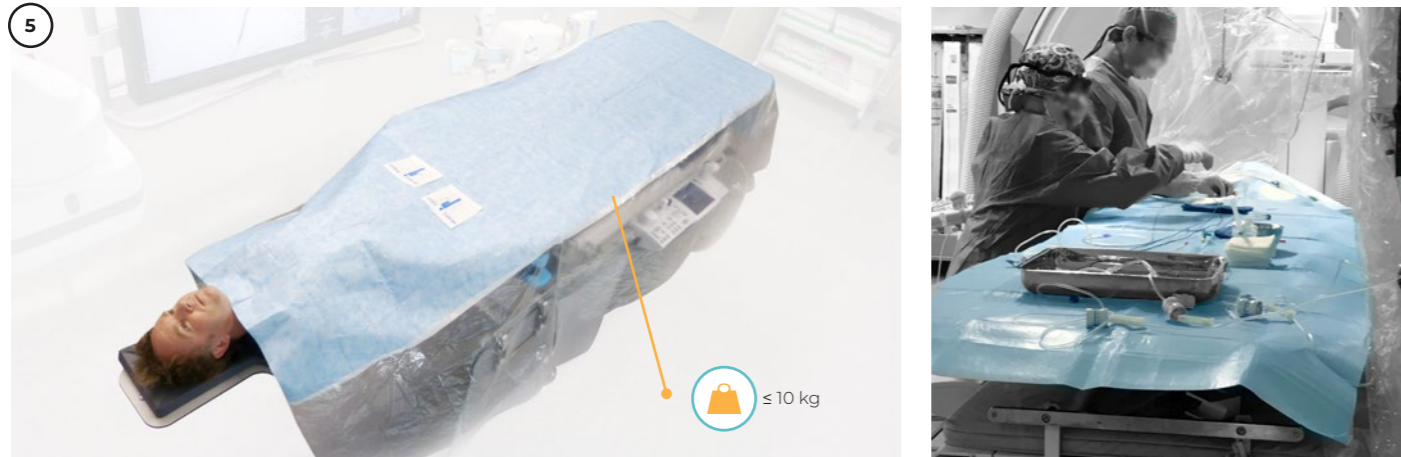Optional Extension (Narrow)

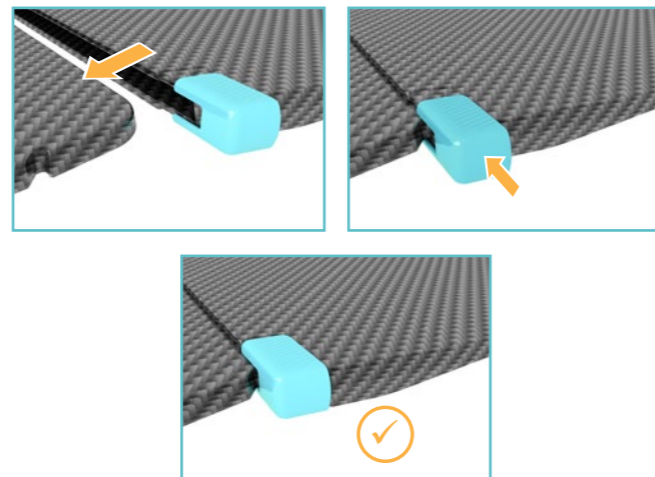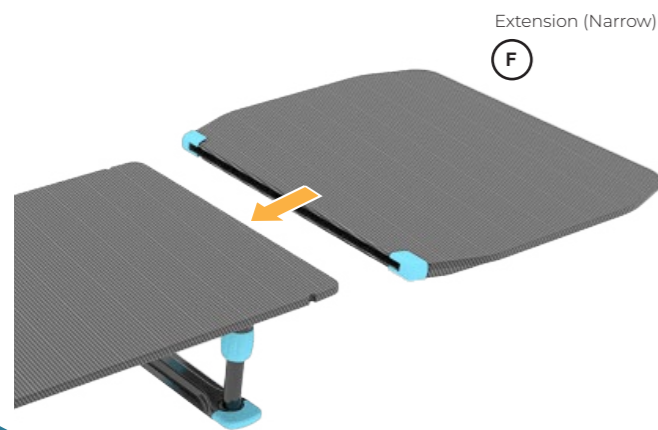Spare Parts

Product Codes - Retrograde IR Platform

	Retrograde IR Platform + Extension	AM0420
	Retrograde IR Platform	AM0400
	IR Platform Extension	AM0410

Component Codes - Retrograde IR Platform

	A	Retrograde Main Platform	M3660
	B	Front Base	M3662
	C	Rear Base	M3663
	D	Leg	M3664
	E	Leg Extension	M3693
	F	IR Platform Extension	AM0410


Product Codes - Retrograde IR Platform - Narrow

	Retrograde IR Platform - Narrow + Extension	AM0460
	Retrograde IR Platform - Narrow	AM0450
	IR Platform Extension - Narrow	AM0470

Component Codes - Retrograde IR Platform - Narrow

	A	Retrograde Main Platform - Narrow	M3650
	B	Front Base - Narrow	M3651
	C	Rear Base	M3663
	D	Leg	M3664
	E	Leg Extension	M3693
	F	IR Platform Extension - Narrow	AM0470

Image Table Verification

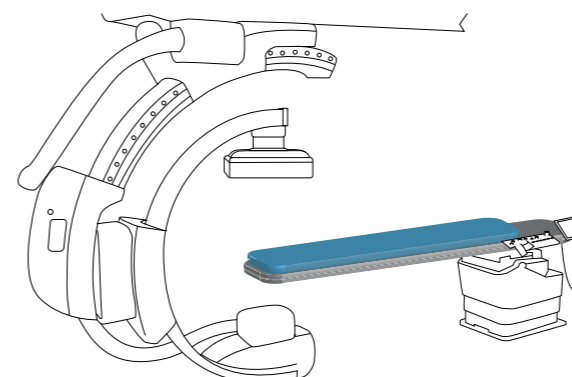The product has been risk assessed to operate within the below specifications

- Table Width:**
AM0420 - AM0400: Retrograde IR Platform ≤ 650 mm
AM0460 - AM0450: Retrograde IR Platform - Narrow ≤ 500 mm
- Table Length:** ≥ 2400 mm
- Table Angulation:** +/- 15° lateral & longitudinal
- Table Surface:** Flat

Essential Information

Information

Intended Purpose

To provide a work surface over a supine patient for femoral transarterial diagnosis and intervention via Retrograde approach on an Angiography imaging table.

Intended User/Training Requirement

Intended to be used by trained medical professionals.

Patient Target Group

Adult male or female patient requiring diagnostic imaging and image guided therapy.

Contraindications

- Not to be used with patients over 135 kg.
- Not for paediatric use.
- Not to be used outside of the recommended image table verification specifications (refer to page 7).

Warnings and Cautions

- Ensure IFU is read prior to use.
- Ensure Preparatory Cleaning is conducted.
- Ensure Preparatory Product Check is conducted.
- Ensure all items placed on the Retrograde Main Platform (Narrow) do not exceed a total weight of 10 kg.
- Do not use with patients over 135 kg.
- Not for paediatric use.

Incident Reporting Guidelines

For product complaints and incidents, please complete form on www.adeptmedical.com/contact/service-and-repairs

Initial Checks & Cleaning

Cleaning

Refer to Disinfection Instructions.

Product Checks

- No exposed Carbon Fibre.
- No visible damage or sharp edges, e.g. cracks.
- Leg: Rotate all four blue locking collars to ensure they are free to rotate and smoothly lock and unlock.
- Leg: Ensure that the Leg is not free to move up and down in the locked position.
- Leg: Ensure that the Legs can move up and down when pushed in the unlocked position.
- Leg: Ensure the Legs are securely assembled on the Retrograde Main Platform (Narrow) (refer to page 4).
- Leg: Ensure Legs are secured in the lower bases (refer to page 5).
- Leg: Ensure Leg is locked during procedure (refer to page 6).
- Extension (Narrow): Ensure the Extension (Narrow) is fully inserted on the Retrograde Main Platform (Narrow) and the Extension clips are fully locked (refer to page 6).

Disinfection

Warnings

- Insufficient cleaning may compromise disinfection process and lead to transmission of pathogens.
- Do not use any disinfectant products or abrasive/corrosive agents which are not on the approved cleaners list.
- Always read manufacturer's instructions and consult the manufacturers MSDS for cleaning and disinfectant products.
- Do not use any cleaning procedure which exceeds 65 °C/149 °F.
- Some disinfectants may cause slight discolouration to the blue material used on some components within the product range. This will not affect the strength and the product will remain fit for purpose.

Limitations on Processing

Discontinue use if:

- Any cracks or breakages are present.
- Any exposed carbon fibre is visible.

Disinfection Instructions

Initial Treatment at the Point of Use

It is important to clean the product once it is removed from the packaging and after each procedure. Ensure all areas including joints, clips, sockets, brackets and levers are thoroughly cleaned after use to remove all contaminant build-up that may be present following a procedure.

Preparation Before Cleaning

- Remove the product from the table.
- Disassemble Retrograde IR Platform (Narrow) into Leg, Leg Extension, Rear Base, Front Base (Narrow), Extension (Narrow) and Retrograde Main Platform (Narrow).
- Ensure the Legs are fully extended.

www.adeptmedical.com/imaging-table-accessories/retrograde-ir-platform/training?group=cleaning-instructions

Cleaning: Manual

- If any visible residue is present, rinse under water, but avoid submerging the device.
- A soft bristle brush can be used.
- Ensure the device is completely dried prior to disinfection.

Disinfection

Refer to the Approved Cleaners List for selection of an appropriate disinfecting agent.

Retrograde Main Platform (Narrow):

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including the threaded holes on the underside.

Extension (Narrow):

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including entirety of slot and around the blue locking clips.

Front Base (Narrow) and Rear Base:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including entirety of slot and around the blue locking clips.

Leg:

- With a cloth dampened by an approved disinfectant, wipe all surfaces.

Leg Extension:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including entirety of the thread.

Approved Cleaners List

Approved Disinfectant by Brands

Liquids:

ORION Laboratories - 70% Isopropyl Alcohol
Jaychem Industries - 2% Chlorhexidine / 70% Alcohol
Betadine - Povidone Iodine (7.5%)
BODE Chemie GmbH - Kohrsolin FF
STERIS - Coverage Spray TB
CaviCide - Metrex Research

Dissolvable:

Du Pont - Rely+On Virkon Tablets

Surface Wipes:

PDI Sani-Cloth Bleach
PDI Sani-Cloth AF3
Reynard Health Supplies - Surface Disinfectant Wipes
Metrex Research - CaviWipes
Clinell Wipes - Universal (Green)
Clorox Hydrogen Peroxide Wipes

Approved Active Disinfectant Ingredients

Acid Based:

≤10% Malic Acid CAS 6915-15-7
≤6% Sulfamic acid CAS 5329-14-6

Alcohol Based:

≤5% 2-Butoxyethanol CAS 111-76-2
≤10% Butyldiglycol CAS 112-34-5
≤70% Isopropyl Alcohol (Propan-2-ol) CAS 67-63-0
≤10% Tridecanol CAS 69011-36-5
<10% Alcohols, C12-14, ethoxylated CAS 68439-50-9
≤70% Denatured Ethanol CAS 64-17-5

Sulphate Based:

≤0.1% PHMB CAS 27083-27-8
≤55% Potassium Peroxymonosulfate CAS 70693-62-8
≤3% Potassium Persulfate CAS 7727-21-1

Ammonium and Chloride Based:

≤5% Benzalkonium Chloride CAS 68424-85-1
≤0.28% Benzethonium Chloride CAS 121-54-0
≤10% Benzyl-C23-18-Alkyl-dimethyl Ammonium Chloride CAS 8001-54-5
≤2% Chlorhexidine CAS 55-56-1
≤10% Didecyl Dimethyl Ammonium chloride CAS 7173-51-5
≤0.5% Quaternary Ammonium Compounds CAS 68956-79-6
<5% Benzyl-C 12-18 alkyldimethylammonium chlorides CAS 63891-01-5

Others:

≤10% Glutaral CAS 111-30-8
≤0.63% Sodium Hypochlorite CAS 7681-52-9
≤7.5% Povidone Iodine CAS 25655-41-8
0.5% Disodium Cocoampho Dipropionate CAS68604-71-7
≤1.4% Hydrogen Peroxide CAS 7722-84-1

Access the latest Approved Cleaners list via www.adeptmedical.com/resources

Drying

All components should be dried thoroughly before use.

Maintenance, Inspection and Testing

For all components ensure:

- No visible damage or sharp edges, e.g. cracks.

Retrograde Main Platform (Narrow), Front Base (Narrow), Extension (Narrow):

- No exposed Carbon Fibre is visible.

For the Extension (Narrow):

- Ensure the Extension (Narrow) can be assembled smoothly onto the Retrograde Main Platform (Narrow) and the locking clips are functional.

For the Front Base (Narrow) and Rear Base:

- Ensure the locking clips are functional and can retain the Leg.

For the Leg:

- Ensure thread is intact and can be screwed onto the Retrograde Main Platform (Narrow).
- Ensure the blue collar rotates smoothly.
- Ensure when blue collar is in the unlocked position the Leg can extend and contract smoothly when force is applied.
- Ensure when blue collar is in the locked position the Leg cannot extend and contract when force is applied.

For the Leg Extension:

- Ensure thread is intact and can be screwed onto the Leg Extension and the Retrograde Main Platform (Narrow).

Packaging

It is not required to package the device following disinfection.

Sterilisation

This device should not be subjected to sterilisation processes.

Storage

Once disinfection is complete and all components are dry the device should be stored in a dry environment in which the temperature will not exceed 65°C/149°F.

Disposal

Used product is a biohazard, decontaminate according to instruction provided in the Disinfection Instructions and reach out to the manufacturer on adeptmedical@adept.co.nz for material specification, if required. Dispose according to internal clinic policy taking into consideration local regulations.

Serious Incidents

Any serious incident which occurs in relation to the device should be reported immediately to:

- The Competent Authority of the applicable Member State.
- The Manufacturer using the Serious Incident Reporting Form found on the Adept Medical Website: www.adeptmedical.com/contact/serious-incident-reporting-form

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Visit the Adept Medical Website for detailed visual instructions related to this device. www.adeptmedical.com

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